

Attention Laundry

WHEN THE SLING BECOMES SOILED OR CONTAMINATED IT CAN BE LAUNDERED WITH MILD DETERGENT IN COLD OR WARM WATER ONLY. DO NOT USE BLEACH AS IT CAN WEAKEN THE STITCHING AND FABRIC.

TUMBLE DRY IN THE DELICATE TEMPERATURE CYCLE IN THE DRYER. IF YOUR DRYER DOES NOT HAVE A DELICATE CYCLE, THE SLING SHOULD BE HUNG TO AIR DRY.

Atencion Launderia

CUANDO EL CABESTRILLO ESTA SUCIO O CONTAMINDO, LAVELO EN DETERGENTE SUAVE, SOLAMENTE EN EL AGUA FRIO O TEMPLADO. NO USE EL BLANQUERO PORQUE PUEDE ENFLAQUECER LAS PUNTADAS O LA TELA.

EN LA SECADORA, SEQUELO EN EL CICLO DELICADO DE TEMPERATURA BAJA. SI LA SECADORA NO TIENE UN CICLO DELICADO, DEBE COLGARLO EN EL AIRE.