

Vera™ Components

- A. Model Name Vera™
- B. Four Point Back Belt Arm The 4 point back belt attachment arm.
- C. Ergonomic Swiveling Hand Grips Ergonomic swivel hand grips for patient comfort and security
- D. Emergency Manual Lowering Device Used to manually lower the lift in the event of power failure
- E. Knee Pad Stabilizes and comfortably supports.
- F. Safety Strap Keeps legs in position and prevents dementia patients from stepping off foot tray.
- G. Foot Tray Stable steel foot plate with non-slip matting. Height adjustable.
- H. Caster 3" casters are hospital grade.
- I. Rear Caster The heavy-duty 4" locking caster is hospital grade.
- J. Auxiliary Up/Down Switch Used to raise/lower the lift in the event of hand control failure.
- K. Base/Battery Box Houses electronics for lift operation including the battery.
- L. Charging Cord Hospital grade charging cord plugs into a hospital grade receptacle or into a standard wall outlet to charge the on-board batteries.
- M. Base Spread Handle Used to open and close the legs of the lift.
- N. Actuator Used to raise and lower the lift arm.
- O. Hand Control Magnetized control used to raise and lower the lift.
- P. Handle Assembly Handles used to maneuver the lift.
- Q. Emergency Stop Switch Used to shut-off power to the lift.



VANCARE®  
We move body and mind

Vera™  
V350, V500, V600, V800



Scan the barcode above with a smart phone for more product information.

VANCARE, INC  
1515 1st STREET  
AURORA, NE 68818  
TEL 800.694.4525  
TEL 402.694.4525  
FAX 402.694.3994  
vancare.com



Made with Pride in the USA

# Protect caregiver and cared-for alike with the versatile Vera™

The Vera™ series of battery-powered lifts allow nursing staff to raise residents/patients from a seated position to standing position with minimal effort. In most cases, the resident/patient can be transferred from a seated position by one staff member, requiring minimal arm or shoulder assistance from the resident/patient.

The Vera™ comes in three weight capacities:

- The Vera™ V350 with a 350 lbs. capacity (page 1)
- The Vera™ V500 with a 500 lbs. capacity
- The Vera™ V600 with a 600 lbs. capacity (page 2)
- The Vera™ V800 with a 800 lbs. capacity (back cover)

A Vera™ lift is a very efficient repositioning aid that reduces injuries among nursing staff, helping to reduce workdays lost to injury, worker's compensation claims, staff turnover, and liability insurance premiums.



V350

# Check out the features that make the Vera™ the brand of choice

- Magnetized hand-held control secures anywhere on the lift
- Auxiliary up/down switch provides alternative means of raising or lowering the resident/patient. Raising In case of hand controller failure and lowering in case of power failures.
- On-board charger assures trouble free operation. The Vera™ is simply plugged into a hospital grade or standard wall outlet to charge.
- Light blue, White Hammertone™, or Vancare Purple™ finish offer years of chip-resistant, non-peeling wear.
- Complies with all applicable federal and state regulations (i.e. OSHA, UL 2601-1, UL60601-1).
- Slip-resistant foot plate is securely mounted and offers two adjustable height positions.
- Knee pad is mounted in the optimum position for the comfort of the resident/patient.



V600

## The versatile Vera™ Series Lift is the ultimate risk manager!

The Dual purpose lift allows for transferring residents/patients via “Sit-to-Stand” transfers for those who are weight bearing and via “Sit-to-Sit” transfers for those who are non-weight bearing.

## World-class slings to meet your needs

Vancare’s sling and belt selection is world class. With several hundred models, sizes and material choices, Vancare can provide the perfect fit for your patients and your budget. Vancare also proudly produces top quality “Made in the USA” products. When your patient’s safety is at stake, you can trust a Vancare sling or belt.

