



AutoSlide is used to aid transfers into and out of car seats. It consists of two sections with surfaces that slide against each other. AutoSlide has a tension strap for fastening at the back. There are handles at the sides for caregivers.

Vancare's products together form the Vancare concept, which takes a holistic approach to patient transfers. Vancare has four categories of transfers: transfer, positioning, support, and lifting.



Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.



Always read the manual


Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.




Functional inspection

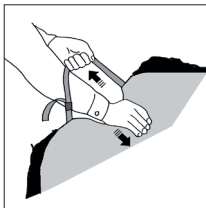

Test AutoSlide's friction by rolling it and noting how easily it glides. If possible, compare with a new AutoSlide.

Test clasp function and handles by applying tension to ensure that the product withstands heavy load.

Placement of AutoSlide

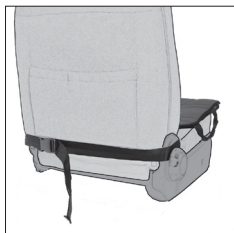

Place AutoSlide with the smaller section's rough surface facing the car seat cushion. Fasten the tension strap around the back support and tighten securely.

Using AutoSlide



The patient can sit on the seat and then easily lift/turn the legs into the car while gliding all the way back onto the seat (1).

A caregiver can assist from the driver's side of the car by pulling on the handles on the sides of the AutoSlide (from the driver's side) (2).

NOTE! The patient should always wear a seatbelt.

Material:

Polyester: Durable material with low friction.

Care of the product:

Read the product label.



Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

Contact your local distributor if you have any questions about the product and its use. See www.vancare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading on our website, www.vancare.com

Manufactured for:

Vancare, Inc.
1515 First St
Aurora, NE 68818

Phone: (800) 694-4525
Fax: 402-694-3994
info@vancare.com

VANCARE[®]
We move body and mind