



SWL:
300 kg/661 lbs

IncoSheet is a washable incontinence barrier that can be used in bed or on a wheelchair seat cushion. The material is highly absorbent and distributes fluid over a large area. The fabric on the top side of the product makes the surface feel dry and comfortable. The rough fabric on the underside provides friction to keep the sheet in place in the bed.

IncoSheet is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.



Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.




Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

Placement of IncoSheet

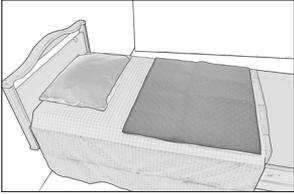

Place the IncoSheet with the quilted, lighter side facing upwards in the bed. IncoSheet is available in four different sizes: two sizes have a shorter incontinence barrier, of which one has fabric on both sides so that it can be bedded in under the mattress, and one size with a longer incontinence barrier. A smaller size, specially developed for placement in a wheelchair, is also available.





75x90 cm

The two shorter incontinence barriers are mainly intended for patients who are able to turn themselves, but they can also be used when the patient requires assistance to turn, in which case a transfer assistive device is placed on top. Since the transfer assistive device is placed on top of the IncoSheet, the IncoSheet remains in the same place under the patient during and after turning, thus continuing to provide incontinence protection.



75x200 cm

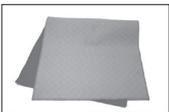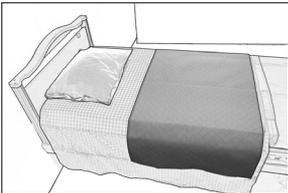

75x150 cm

IncoSheet with the longer incontinence barrier is mainly intended for patients who need help with turning. This size model can be placed on top of the transfer assistive device, since it has an extra long incontinence barrier and continues to provide protection, even after turning.



40x83 cm

The small model, IncoSheet 40x83, can be placed in a wheelchair to protect the seat cushion.

Note!


IncoSheet is not designed to be used as a draw sheet, but is instead bedded over a bed sheet, a draw sheet or other transfer assistive device.

Material:

Polyester, cotton

Care of the product:

Read the product label.

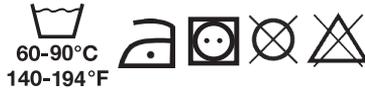

IncoSheet can be laundered and dried at high temperatures. Do not use laundry rinse or bleach (e.g., chlorine). IncoSheet withstands more than 100 launderings.

Size:

1530	75 x 90
1531	75 x 200 with fabric on both sides
1532	75 x 150
1533	40 x 83

Contact your local distributor if you have any questions about the product and its use. See www.vancare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.vancare.com.

Manufactured for:
Vancare, Inc.
1515 First St
Aurora, NE 68818
1-800-694-4525
www.vancare.com
info@vancare.com