

Art.no	Height
8090	10 cm
8095	5 cm

FootStool is used to raise the position of the knees and thereby enable safer, easier bed-to-wheelchair transfer in a seated position. FootStool also provides good support under the feet during visits to the toilet, or when repositioning farther back in a chair. FootStool is made of a material with closed cells, which makes it easy to keep clean.

FootStool is part of a series of different models of transfer assistive devices. The choice of model and material is determined by the patient's functional capacity and the situations in which the product is being used.

Vancare's products together form the Vancare concept, which takes a holistic approach to patient transfers. Vancare has four categories of transfers: transfer, positioning, support, and lifting.



Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.




Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.


Do not leave the patient unattended during a transfer situation.

Placement of FootStool

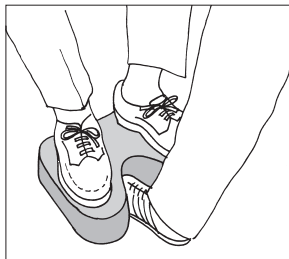

Place the FootStool under the patient's feet when the patient is sitting on the toilet or in a wheelchair. FootStool is recessed at the front to allow the caregiver to place a foot as close as possible to the stool. The back also has a small recess to facilitate placement close to the toilet.

Using FootStool

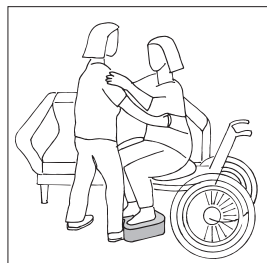

FootStool complements other assistive devices such as EasyBelt/FlexiBelt, EasyGlide and SitWalk to facilitate transfers between two locations or for repositioning, for example, farther back in a chair. Since the knees are raised, the thighs are also lifted somewhat, which reduces friction and makes the transfer easier.



Material:

Polyethylen

Care of the product:

Clean with dishwashing liquid or disinfectant. Wipe with a cloth.

Note: Do not subject the product to hard pressure or impact. This may leave indentations in the material.

Contact your local distributor if you have any questions about the product and its use. See www.vancare.com for a complete list of distributors.

Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website,

www.vancare.com

Distributed by:
Vancare, Inc.

1515 First St
Aurora, NE 68818

Phone: (800) 694-4525

Fax: 402-694-3994

info@vancare.com

VANCARE[®]
We move body and mind