


MiniStretcher

VANCARE®
We move body and mind



Manual - English

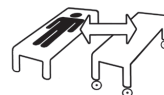

Max: 200kg/440 lbs

MiniStretcher is used in emergency care units and ambulance services for lifting and transfer. Transfers between two locations, for example, bed to stretcher or lifting from the floor. MiniStretcher is easy to keep clean and store.

MiniStretcher is part of a series of different models of transfer assistive devices. The choice of model and material is determined by the patient's functional capacity and the situations in which the product is being used.

Functional inspection

Visual inspection



Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Always read the manual

Always read the manuals for all assistive devices used during a transfer.
Keep the manual where it is accessible to users of the product.
Do not leave the patient unattended during a transfer situation.





Mechanical load

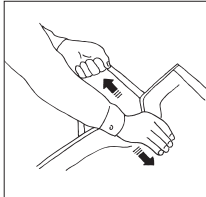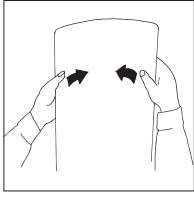

Carefully bend the MiniStretcher to check strength and durability. The mat should be both elastic and rigid. Discard the MiniStretcher when its strength and durability are diminished.


Pull on the handles to ensure that they withstand a heavy load. If there are signs of wear, the product must be discarded.

Placement and use of MiniStretcher

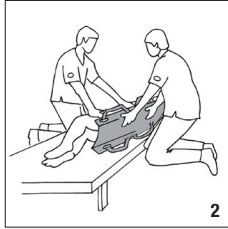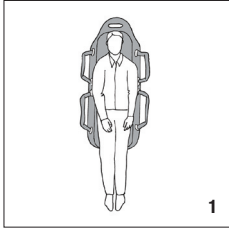

1. Place the patient centred on the MiniStretcher and ensure that there is sufficient support for the head.

2. Turn the patient on one side and fold the handles under the MiniStretcher before it is pushed in under the patient.

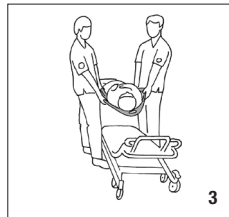

3, 4. Work in pairs (or more) one on either side, or one at each end, and lift the patient. Plan the transfer so as to minimize time taken, lifting and distance.

Storage and cleaning



Keep the MiniStretcher on an ambulance stretcher, as shown in the picture.

MiniStretcher should be cleaned with a pressure washer and disinfectant.

Material:

Polyester

Contact your local distributor if you have any questions about the product and its use. See www.vancare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.vancare.com

Vancare, Inc.
1515 First St
Aurora, NE 68818

Phone: (800) 694-4525
Fax: 402-694-3994
info@vancare.com

VANCARE[®]
We move body and mind