



Max: 150 kg/330 lbs  
200 kg/440 lbs

OneWaySlide is used for repositioning farther back in a chair/wheelchair. OneWaySlide is designed to glide easily backward, but not forward. To make manoeuvring easier for caregivers, OneWaySlide features handles on both sides and at the back. It is placed so that the patient remains sitting on it. OneWaySlide is available in several sizes.

Vancare is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.



## Functional inspection

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.




## Always read the manual


Always read the manuals for all assistive devices used during a transfer.  
Keep the manual where it is accessible to users of the product.  
Do not leave the patient unattended during a transfer situation.



## Functional inspection

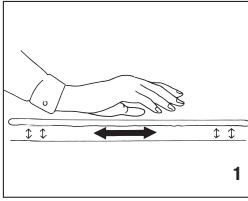


Test OneWaySlide's friction by rolling it and noting how easily it glides. If possible, compare with a new OneWaySlide.

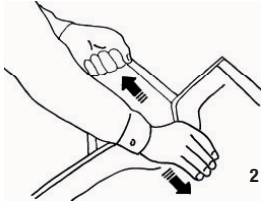


Test the seams and strength of the handles by pulling the handles in different directions.

## Placement of OneWaySlide

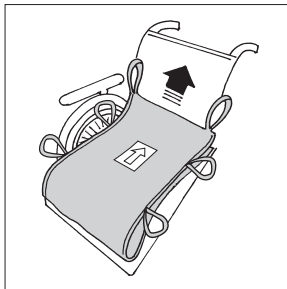


Place OneWaySlide so that the open section is facing the back of the wheelchair. Place the upper section so that it sticks up a bit on the backrest of the wheelchair. This makes it easier for the caregiver to reach the handles.

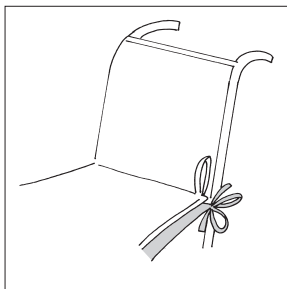


Tie the OneWaySlide to the rear frame tubing of the wheelchair.


## Alternative grips

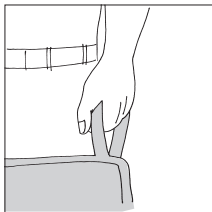


The caregiver can either simply grasp the handle (1), or first loop it around the wrist (2). Remember to work with knees bent, back and arms straight, and shift the weight.

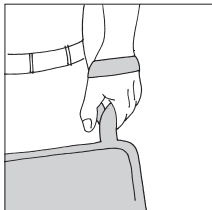

## Using OneWaySlide

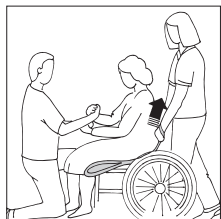

1. Positioning is easier if the patient can lean forward. If the patient is unable to lean forward, one caregiver can assist from the front while supporting the patient's balance.

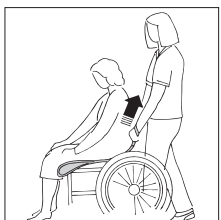

2. If the patient can lean forward and has good balance while seated, only one caregiver is required. Grasp the appropriate handles and pull the patient back by drawing the OneWaySlide upward along the backrest of the wheelchair.



3. If the patient is heavy or if the chair has a high back, two caregivers can assist, one working on either side. Coordinate your efforts.

Raising the patient's feet from the floor will relieve some of the weight of the thighs from the seat, thereby making repositioning easier.

## Material:

Polyester

## Care of the product:

Read the product label.



Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

## Size and safe working load:

2074	110 x 30 cm	150 kg/330 lbs
2075	130 x 40 cm	150 kg/330 lbs
2076	130 x 45 cm	150 kg/330 lbs
2077	130 x 55 cm	200 kg/440 lbs

Contact your local distributor if you have any questions about the product and its use. See [www.vancare.com](http://www.vancare.com) for a complete list of distributors.

Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.vancare.com](http://www.vancare.com)

Vancare, Inc.  
1515 First St  
Aurora, NE 68818

Phone: (800) 694-4525  
Fax: 402-694-4525  
[info@vancare.com](mailto:info@vancare.com)