


# TurnTable

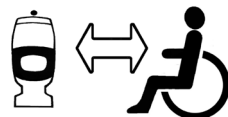

**Max:**

5082: 135 kg/297 lbs

5083: 135 kg/297 lbs

5084: 250 kg/551 lbs

5085: 135 kg/297 lbs

The TurnTable can be used for seated transfers between wheelchair and bed, toilet or chair/armchair. For increased security, the TurnTable can be used together with FlexiBelt/EasyBelt. TurnTable is available in three sizes with handles or without handle for tight spaces.

The TurnTable is part of a series of different models of transfer assistive devices. The choice of model and material is determined by the patient's functional capacity and the situations in which the product is to be used.



## Functional inspection

### Visual inspection

Check the condition and function of the assistive device regularly. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.



## Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

## Special requirements and conditions

To transfer with TurnTable, the patient must be able to:

- support weight while standing
- stand up and maintain balance (with support)
- understand instructions

## Placement and use TurnTable



Place the TurnTable on the floor beside the patient. Secure the TurnTable in place, before raising the patient, by placing the ball of your foot on the turntable and your heel on the floor. Steer the TurnTable by pushing with the front of the foot and braking with the heel. When the patient is about to sit down, secure the TurnTable in place as described above.



When transferring from bed to wheelchair, place the wheelchair close to the bed, so as to minimize transfer distance.



When transferring to the toilet, place the wheelchair close to the toilet, so as to minimize transfer distance. Pull the patient's pants up/down when the patient has risen slightly from the seat. Preferably, two caregivers should assist.

## Care of the product:

Clean TurnTable with pressurized air or water. If necessary, disinfect with 70% alcohol. Teflon spray or a similar product can be used to lubricate between the plates so that they glide easily against each other.

### Size

5082	30 cm in diameter
5083	38 cm in diameter
5084	45 cm in diameter, with handle
5085	38 cm in diameter, with handle

Contact your local distributor if you have any questions about the product and its use. See [www.vancare.com](http://www.vancare.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.vancare.com](http://www.vancare.com).

Manufactured for:

Vancare, Inc.  
1515 First St  
Aurora, NE 68818  
1-800-694-4525  
[www.vancare.com](http://www.vancare.com)  
[info@vancare.com](mailto:info@vancare.com)